



SARS-CoV-2 Antigen Rapid Test Kit (Colloidal Gold)

TEST METHOD

SARS-CoV-2 Antigen Rapid Test Kit (Colloidal Gold)



1. Get the test device, buffer, extraction tube, swab, test tube rack and timer ready to use.

SARS-CoV-2 Antigen Rapid Test Kit (Colloidal Gold)




2. Twist off the top of the buffer bottle, slowly dispense all of the buffer into the extraction Tube.

SARS-CoV-2 Antigen Rapid Test Kit (Colloidal Gold)




3. Insert the swab into both nostril of the patient. The swab tip should be inserted up to 2.5 cm (1 inch) from the edge of the nostril. Roll the swab 5 times along the mucosa inside the nostril to ensure that both mucus and cells are collected.

SARS-CoV-2 Antigen Rapid Test Kit (Colloidal Gold)

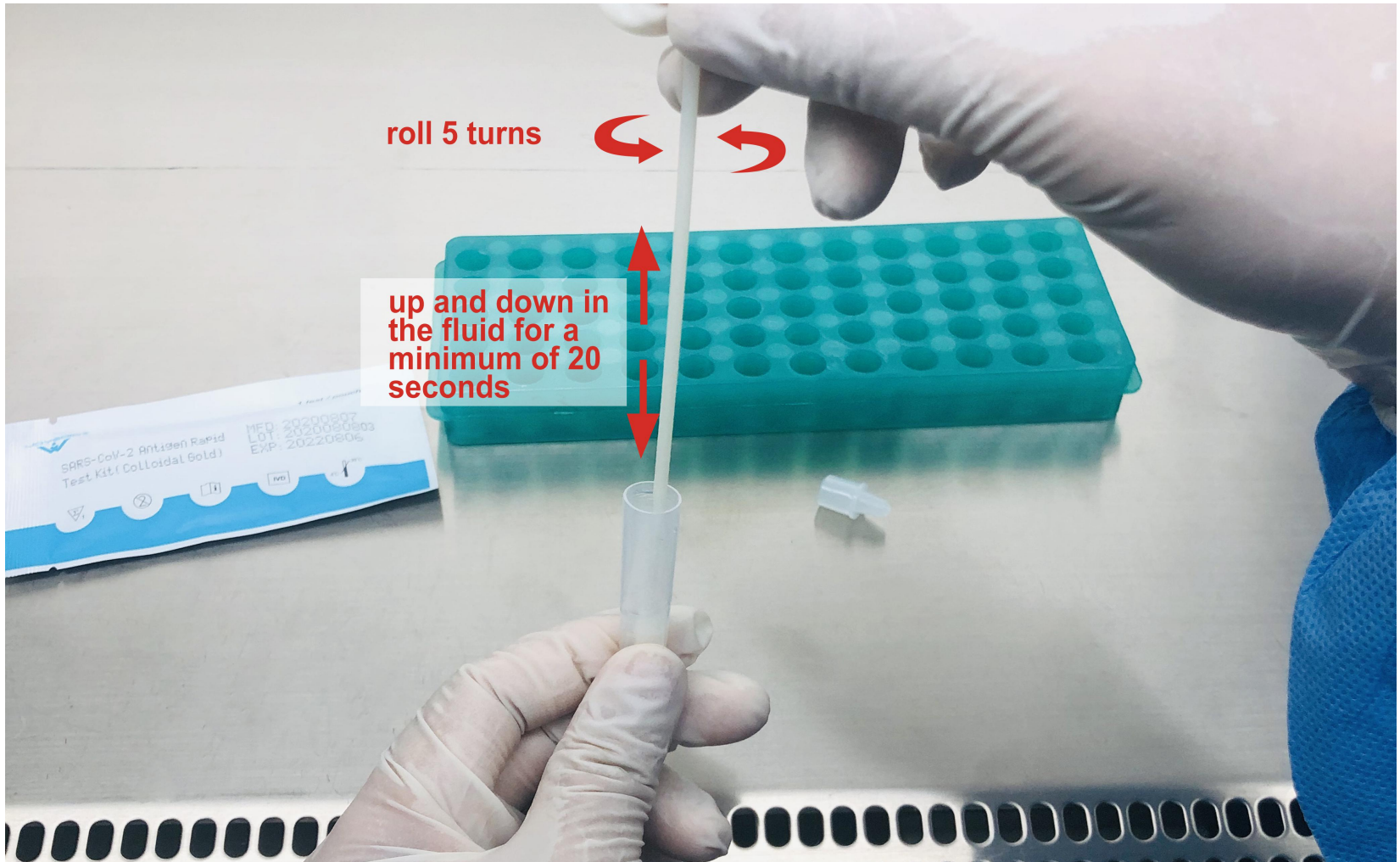


4. Insert the swab into the tube and plunge the swab up and down in the fluid for a minimum of 20 seconds, then hold the swab against the bottom of the tube and roll 5 turns.

SARS-CoV-2 Antigen Rapid Test Kit (Colloidal Gold)

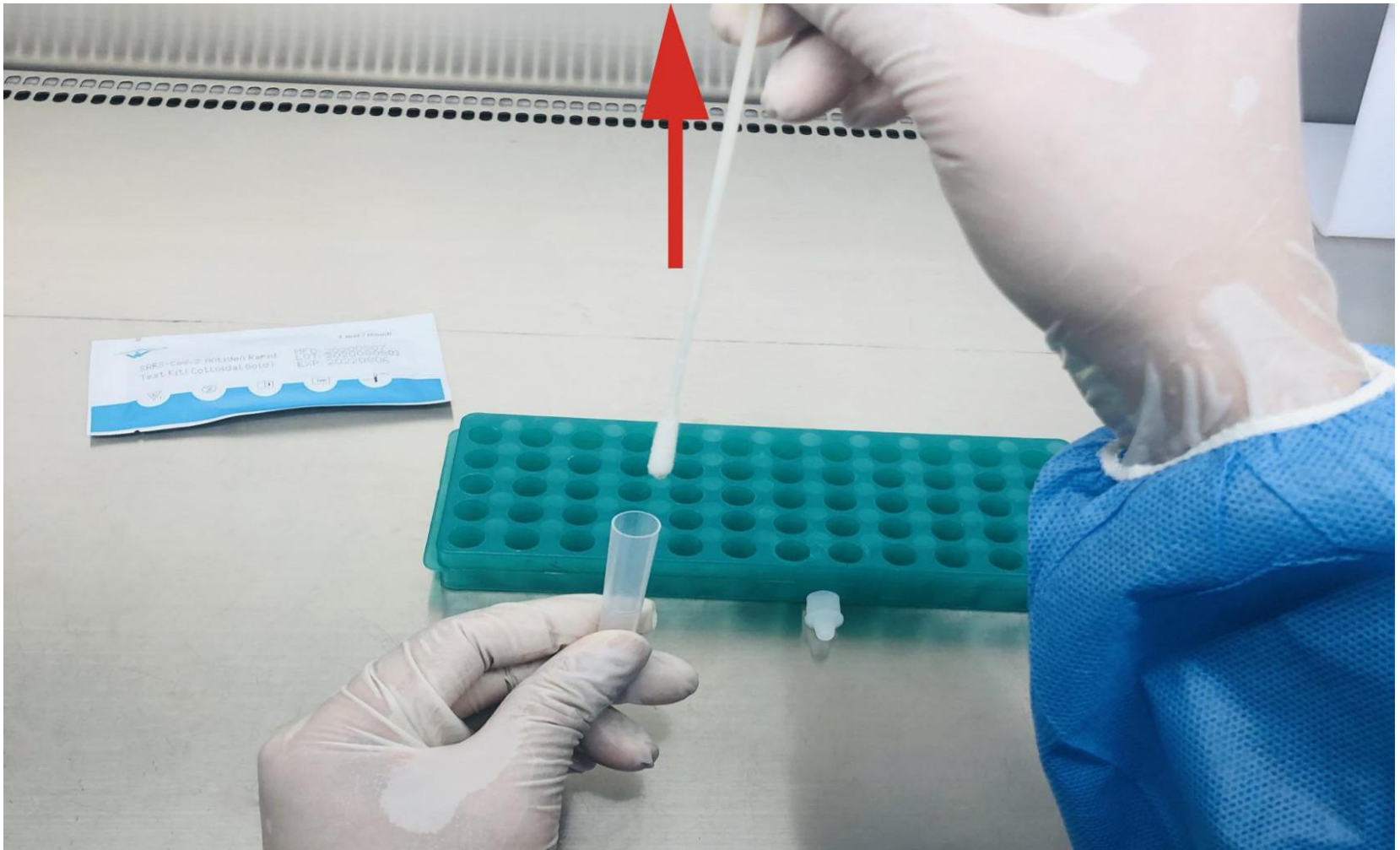

5. Remove the swab.

SARS-CoV-2 Antigen Rapid Test Kit (Colloidal Gold)

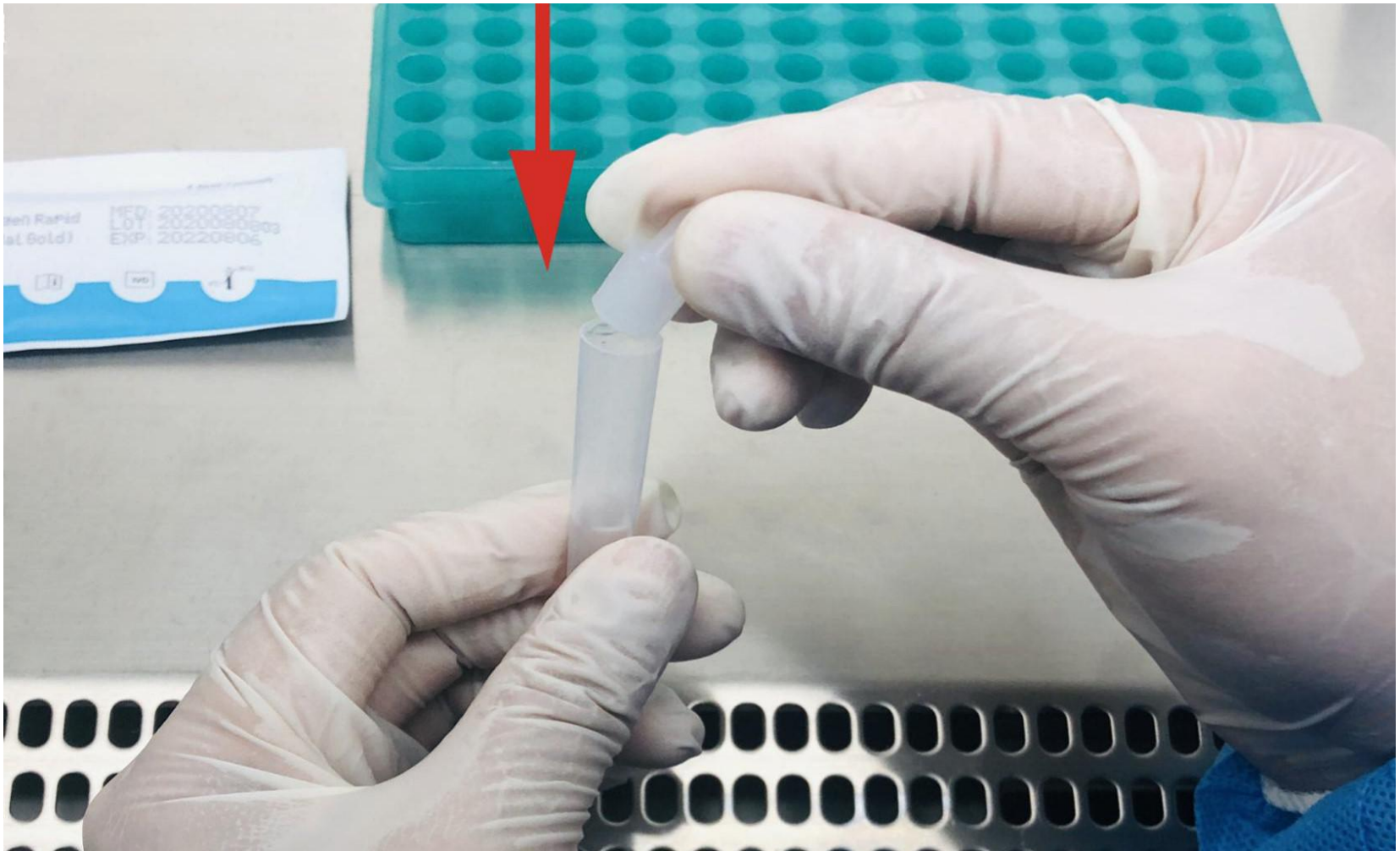

6. Press the nozzle cap firmly onto the extraction tube and place the tube in a rack in the designated area of the workspace.

SARS-CoV-2 Antigen Rapid Test Kit (Colloidal Gold)



7. Tear off the foil pouch, take out the test cassette and place the cassette on a clean and level surface.

SARS-CoV-2 Antigen Rapid Test Kit (Colloidal Gold)




8. Gently squeeze the ridged body of the tube, dispensing three (3) drops of the processed specimen into the sample well.

SARS-CoV-2 Antigen Rapid Test Kit (Colloidal Gold)



9. Read the test results between 15 and 20 minutes.

SARS-CoV-2 Antigen Rapid Test Kit (Colloidal Gold)

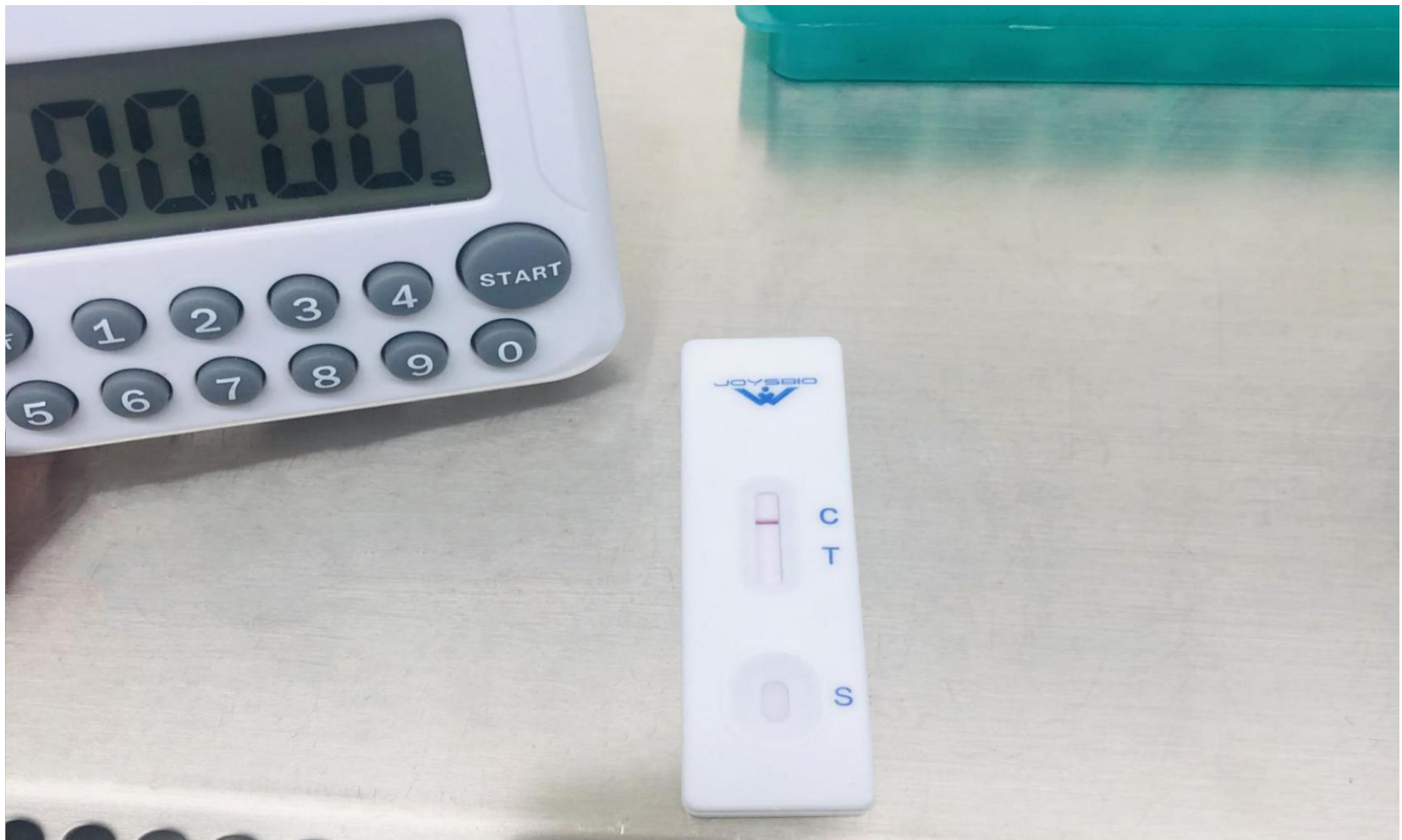

10. POSITIVE: Two lines appear. One colored line should be in the control line region (C), a colored line appears in test line (T) region. NEGATIVE: Only one colored control line appear. INVALID: Control line fails to appear.



JOYSBIO (Tianjin) Biotechnology Co., Ltd.